

INSTILLAGEL®

STERILE LUBRICANT lidocaine 2%

INSTILLAGEL® LIDO

- Excellent lubrication properties of hydroxymethylcellulose base adheres especially well to the urethra.
- 11ml syringe for that extra bit of lubrication.
- Presented in a sterile blister pack suitable for theatre or ward use.
- Syringe specifically designed for urethral instillation.
- Does not fit a parenteral needle minimising risk of I.V. injection.

INSTRUCTIONS FOR USE

- Peel open Instillagel® sterile blister pack and remove the Instillagel syringe.
- Prime plunger by either pushing or pulling slightly on the plunger handle.
- Remove syringe cap and press plunger lightly to coat syringe tip with a small amount of gel prior to insertion.
- Insert syringe into urethra up to the syringe shoulder and slowly press plunger to instill gel into urethra.
- Remove syringe and wait approximately 5 minutes for the full anaesthetic effect.

OPTIONAL INSTILLATIP®

Flared nozzle attachment available for Instillagel® syringes, presented in a separate sterile pack.



INSTILLAGEL®

STERILE LUBRICANT

lidocaine 2%



A UNIQUE FORMULATION TO SUIT YOUR USE

Viscosity designed to spread more easily up the urethra and coat the entire surface.



STERILE PACKAGING THAT CONFORMS TO ASEPTIC TECHNIQUE

Instillagel® comes in an autoclaved blister pack for aseptic technique procedures without any assembly of the syringe required prior to use.



SYRINGE TIP SPECIFICALLY DESIGNED FOR URETHRAL INSTILLATION OF GEL

Fits easily into the urethra with the base of the syringe acting as a stopper against backflow during delivery. The tip is highly polished and smooth. It won't cause damage to the meatus or cause unnecessary discomfort when the syringe is inserted and positioned correctly.

Distributed in Australia by: InterPharma Pty Ltd
Suite 103 / 39 East Esplanade, Manly NSW 2095 Ph: 02 9976 6876 www.interpharma.com.au
Distributed in NZ by: Healthcare Logistics
58 Richard Pearse Drive, Airport Oaks, Mangere 2022 Ph: 09 918 5100

InterPharma